

TAKING A 24-HOUR HISTORY AT 0800 HRS EACH DAY

The Patient Total = Volume Infused + Bolus Given.

1. At the same time each day, complete a 24-hour patient total. Do this by pressing the blue **info** key twice.
2. Check the VTBI (remaining volume) by only pressing the blue **info** key twice. Check the bolus summary by repeated pressing of the blue **info** key.

TAKING AN 8-HOUR HISTORY

The Patient Total = Volume Infused + Bolus Given.

1. Press the red **stop/no** button.
2. Press the blue **info** button twice to obtain the 8-hourly screen. **THIS IS NOT A RUNNING TOTAL.**
3. Press the red **stop/no** button twice, then press the green **start/ok** button to resume the infusion.
4. Check VTBI (remaining volume) by only pressing the blue **info** key twice.
5. At the same time each day, complete a 24-hour patient total by following step 4, and complete the 24-Hour Patient Total Summary. This is a cumulative total, so you will need to subtract the previous 24-hour total.